

Montagehandleiding accu Nortwood

1. Mount the plastic socket with the Spring cable on the spool in the specified holes and tighten with screws.



2. Remove the plastic fasteners from the battery and fasten the triangle part to the battery with the same screws as the plastic fasteners.




3. Fasten the triangle part to the clamp with an screw M8 .




4. To fix the battery on the mechanism by means of a collar having preliminary established a detail and to tighten the screw.



5. Connect the cables and connectors as shown in the photo so that you can unwind the common harness when using a battery connection and secure with two ties (1). **Pay attention to the length of the cables in the photo !!!**

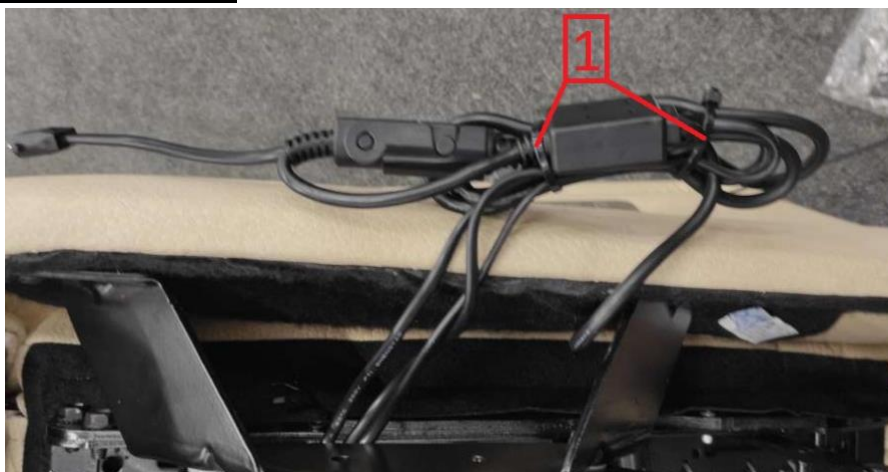

6. If the battery is connected, turn the power cable to the front of the chair, connect it to the battery and secure it with the ties (1), and additionally with the textile tape (2).

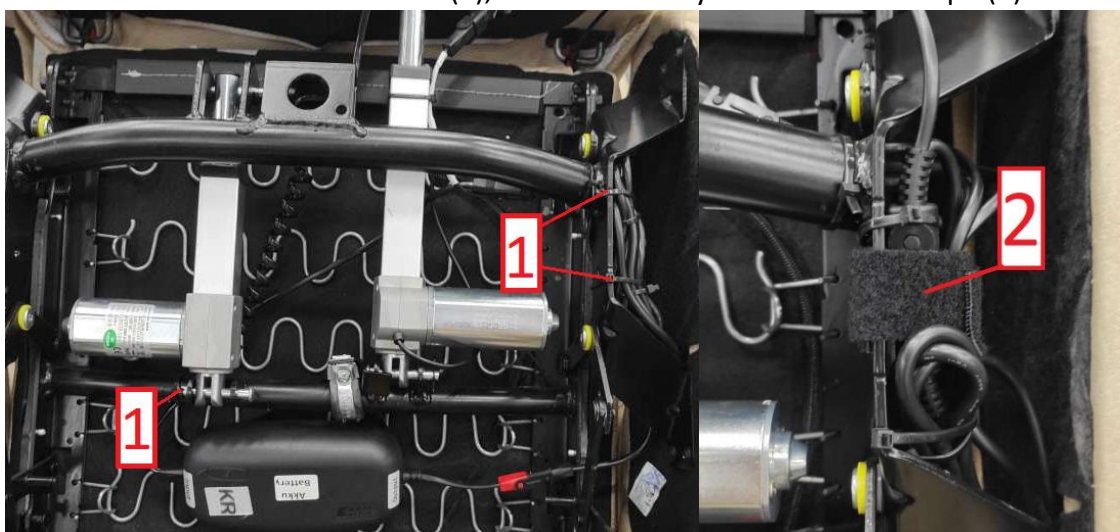

7. Connect the cables to the battery **Spring cable>Input** and **Output> Superior branching cable**
